

Cleaning and care instructions

for SHERAprint-splint taff & flex



Cleaning and care instructions



Dear patient,

To ensure that you can enjoy your splint, repositioners, mouthguard or nightguard for a long time, we recommend:

Daily cleaning

Use lukewarm water, a soft toothbrush or liquid soap for daily cleaning. Rinse well afterwards.

Advice

- Do not use toothpaste, the abrasive particles are coarse and scratch the surface.
- Do not use acids or solvents.
- Do not use mouthwash



SHERA Werkstoff-Technologie GmbH

Espohistr. 53 | 49448 Lemförde | Germany
Tel.: +49 (0) 5443-9933-0 | www.shera.de